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Product Introduction

Acellular Dermis is processed from donated human allograft. This freeze-dried membrane is comprised of highly purified type I collagen fibers which are bioresorbable, so it needs no removal surgery. This membrane has mechanical strength and tearing resistance. Because of flexibility & adhesive properties, it can be easily adapted to the defect size.

Biocompatibility

It is biocompatible and compatible with surrounding blood cells. It is pH balanced-identical pH of blood in humans; It is suitable for stem cell and fibroblast seeding.

Processing Method & Sterilization

After screening healthy donors with age limitation (more than 13 years old) by evaluating medical history and physical examination, we have quality control tests with microbiologic & serologic tests. Each sample is tested for aerobic and anaerobic bacteria, fungi & HIV antibodies type 1, 2 (anti- HIV1&2), antibody to human T-lymphotropic virus type 1,2 (HTLV Ab1,2), Hepatitis B core antibody (HBcAb), Hepatitis C virus antibody (HCVAb), Hepatitis B surface Antigen (HBsAg), & RPR for syphilis. The confirmation of HIV, HCV and HBV PCR tests is also mandatory. Before cleaning & processing, final validation has been carried out. All procedure is done under the clean conditions of the class B cleanroom at I.T.P Co. After complete processing and packaging procedure, we also use Gamma irradiation for final sterilization.

Indications for Use

It is used as a scaffold in dentistry, burns, periodontal, maxillofacial surgery for GBR & GTR, E.N.T, reconstructive surgery.

Contraindications

Do not use product when the necrotic tissue, active or latent infection is in or around the surgical site. Do not use this product on unknown defects too.

Side Effects

There is a possibility of infection, inflammation, adverse reaction and rejection.

Presentation and Packaging

Acellular Dermis packed in sterilized single use package and has been presented for single patient use. The product has been packed in double pouches, the first pouches from double film that placed in film and medical grade paper.

Warning and Precautions

Do not use this product if:

- ✓ The expiration date on the package is passed.
- ✓ Any of the package or product appears to be missed, discolored or damaged.
- ✓ The product label is damaged and missing.

Caution

Please avoid resterilization the product.

Please use the contents of any package for only one patient.

Please report any adverse effect attributable to this product to us as soon as possible.

Contact Number: +98 9024264707

Storage

Store this product at ambient temperature 15-30 degree centigrade. No refrigeration or freezing is needed. Acellular Dermis products have 5 years shelf life.

Preoperative preparation

Preparation of the graft bed is important for graft in corporation and new tissue formation. Sterile technique must be maintained to minimize the risk of postoperative complications. The amount of graft material needed is based on the defect size and the procedure & is determined by the judgment of the clinician.

Instructions for Use

- ✓ Peel open outer package (nonsterile).
- ✓ Using sterile technique, transfer contents to a sterile field.
- ✓ Soak Acellular Dermis in a suitable physiological serum or recipient's blood for about 15-20 minutes. (mandatory)
- ✓ Put the combination in the surgical site.
- ✓ Discard any unused portion.

Health Insurance Products

- ✓ Implementation of instructions for product preparation and processing in compliance with the requirements of the Food and Drug Administration of Iran and the National Medical Device Directorate.
- ✓ Observance of GMP production principles
- ✓ Obtaining the international standard for medical devices- Quality management systems ISO 13485
- ✓ Receiving the product license and originality from National Medical Device Directorate labels from the Food and Drug Administration of Iran.
- ✓ Supervising the quality control system In-House and supervising the competent institutions of the Ministry of Health and medical education on preparation, production and distribution.

Manufacturer:

Address: Iranian Tissue Product Co.

Danesh 2 St., Phase 2, Pardis Technology Park, 20th km Damavand Road, Tehran, Iran.

Telefax: +9821 68105

E-mail address: info@regen.ir

References

- The 14th Edition of American Association of Tissue Banks Standards for tissue banking (AATB)
- European Good Tissue & Cell Practices (Euro GTP)